

Rystora[®]

IN HARMONY WITH YOUR SKIN



Adhesive Remover– Spray (Non-Sterile)



FAST AND EFFICIENT REMOVAL OF
ADHESIVES, REDUCE THE SKIN DAMAGE

Rystora® Adhesive Remover – Spray

uses medical silicone as the base to help removal of the sticky skin barrier in medical grade (such as stoma skin barrier, tape and dressing) and adhesive residues. Hope to reduce the skin damage and epidermal peeling caused by removing the sticky skin barrier the pain. Because of the alcohol-free formula, there will be no tingling sensation caused by alcohol.



- No Sting & Alcohol-Free

- Biocompatible

- Feels Moisturizing




- After Use and Does Not Cause Dry Skin.

- Quick Action in Seconds


DIRECTIONS FOR USE

Removing the sticky skin barrier for medical grade stoma skin barrier:

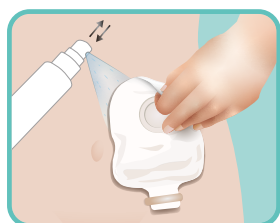


1. Lift up edge of the stoma skin barrier and apply Adhesive Remover- Spray under the dressing while lifting off. Tear the ostomy skin barrier as spraying the Adhesive Remover- Spray until the stoma skin barrier is completely peeled off.
2. After wiping clean the surrounding skin area, stick a new stoma skin barrier.

Removing the sticky skin barrier for medical grade tape, dressing, or adhesive residues:

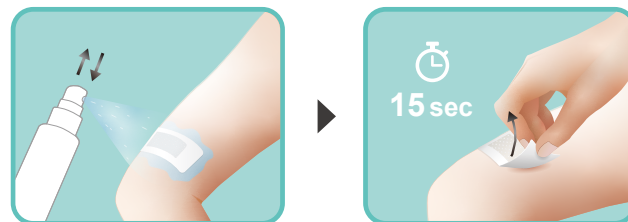

Spray Adhesive Remover-Spray on the sticky skin barrier or adhesive residues. Stay 15 seconds, and tear off or wipe off. If necessary, you can use a new sticky skin barrier.

Removing the no sting barrier film:

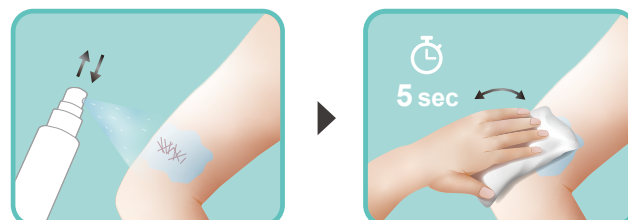

Spray Rystora® Adhesive Remover-Spray on area of no sting barrier film. Stay for 5s, and then wipe clean the area.

SPECIFICATION

REF. No.	Description
SR-085	Spray bottle 85 ml Non-Sterile

Rystora®

Fortune Medical Instrument Corp.

6 F., No. 29, Sec. 2, Jhongjheng E. Rd., Danshuei Dist,
New Taipei City 251, Taiwan

Manufactured by: No. 256, Changchun 2nd, Rd.,
Jhongli Dist, Taoyuan City 320, Taiwan

T. +886 2 2624-2233 www.rystora.com

Rystora® is Registered Trademark of Fortune Medical.

EDM ADHESIVE REMOVER SPRAY Eng. V20221006