

Rystora®

IN HARMONY WITH YOUR SKIN



Silicone Scar Gel

(Non-Sterile)



EFFECTIVE TREATMENT FOR SCARS AT
FACE, JOINTS OR IRREGULAR AREAS

Rystora® Silicone Scar Gel is a safe, effective treatment for scars and keloids at the face, joints or irregular areas and other flat skin. Rystora® Silicone Scar Gel enhances skin rejuvenation following recent surgery, scalds, burns or minor skin injuries, as well as effectively reduces the size, hardness, itchiness and discoloration of scars.


Scar healing principle: Rystora® Silicone Scar Gel can easily be applied over the scar as a thin film, no sticky, self-dry and no allergic reaction to the skin, which keeps moisture on the skin surrounding the scars a protective film, reduces the formation of collagen, makes skin softer and restores scar more quickly.

- Suitable for any irregular scars
- Apply directly, don't need waiting for drying
- Reduce scar hyperplasia
- Reduce discoloration of scars
- Suitable for sensitive skin
- Moisture and breathable

SPECIFICATION

REF. No.	Description
SGA01501	15 ml Non-Sterile

DIRECTIONS FOR USE

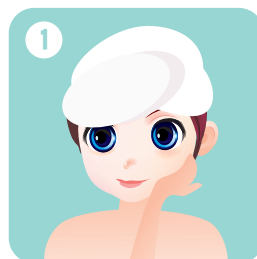

1. Ensure that the scar or skin is clean and dry, and free of ointment or cream before applying.



2. Squeeze the right amount of gel.



3. Apply the scar one-way along to completely cover the scar and form a silicone protective film. The recommended minimum treatment is 2-3 times daily.

Larger and older scars may take minimum of 2 months of treatment. Red and darker scars may require continued use beyond 12 months. Continue using Rystora® Scar Gel after scar restoration for more effective results.

Rystora®

Fortune Medical Instrument Corp.

6 F., No. 29, Sec. 2, Jhongjheng E. Rd., Danshuei Dist,
New Taipei City 251, Taiwan

Manufactured by: No. 256, Changchun 2nd, Rd.,
Jhongli Dist, Taoyuan City 320, Taiwan

T. +886 2 2624-2233 www.rystora.com

Rystora® is Registered Trademark of Fortune Medical.

EDM GEL Eng. V20220906