

# ***Rystora***<sup>®</sup>

IN HARMONY WITH YOUR SKIN



## ***Skin Protector*** ***– Spray*** (Non-Sterile)



PROTECT MINOR INJURIES  
& FRAGILE SKIN

---




Rystora® Skin Protector Spray can quickly create a transparent, breathable, waterproof, quick-drying and non-sticky protective film on the skin surface. The film helps the skin's minor cuts and abrasions, or cracked mound, and prevent the risk of medical adhesive related skin injury to create a breathable and waterproof barrier to avoid the irritation of damaged skin caused by friction or be caused pain by contact with irritant. Good characteristic includes:

- Protect Minor Injuries & Fragile Skin
- Prevent the medical adhesive related skin injury (MARSI)
- Alcohol-free & no sting
- Odorless & tasteless & Transparent
- Quick drying 5 secs after spray
- No stickiness after drying & will not stick to clothes
- Waterproof & breathable
- Biocompatible


## SPECIFICATION

REF. No.	Description
SY01030	Spray bottle 30 ml Non-Sterile

## DIRECTIONS FOR USE

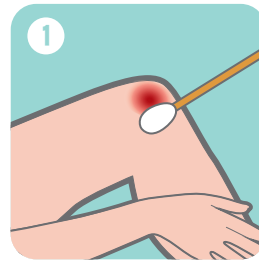

1. Clean and thoroughly dry the skin area of spray protective film.

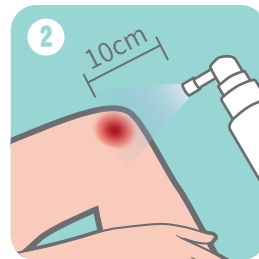

2. Shake well before use. Hold spray nozzle 10cm to 15cm (4 inch to 6 inch) from the skin. Apply an even coating onto the skin.



3. Need to wait for about 5 seconds has blow-dried completely before spraying the second layer. Repeat sprays three-layer.



4. Repeat sprays three-layer protective film onto the Injured skin is recommended every 4 hours.

# Rystora®

**Fortune Medical Instrument Corp.**

6 F., No. 29, Sec. 2, Jhongjheng E. Rd., Danshuei Dist,  
New Taipei City 251, Taiwan

Manufactured by: No. 256, Changchun 2nd, Rd.,  
Jhongli Dist, Taoyuan City 320, Taiwan

T. +886 2 2624-2233    [www.rystora.com](http://www.rystora.com)

Rystora® is Registered Trademark of Fortune Medical.

EDM SPRAY Eng. V20210625