

Rystora[®]

IN HARMONY WITH YOUR SKIN




Silicone Scar Tape ***(Non-Sterile)***



FLATTEN AND FADE SCARS &
PREVENT FORMATION OF SCARS



Rystora® Silicone Scar Tape is used to flatten and fade the new and old scars, and help to prevent the formation of scars including hypertrophic scars and keloids. Scar Tape can be repositioned, no sticky hair, no residual glue. Without disrupting fragile skin layers or causing patients undue pain. Soft translucent blends with skin color and fits better with the skin. The cutting line is designed to neatly torn by hand to the length you need. Adhesive performance is good, waterproof and breathable.

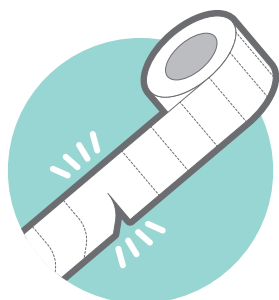


Cutting line



Used on joint skin



Waterproof

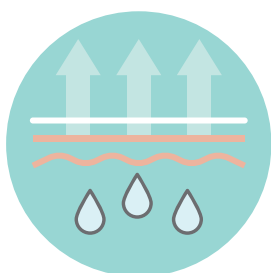

Moisturizing & Breathable

- Made of polyurethane film and Silicone gel.
- Cut-line design, easy to use and torn neatly.
- Pre-cut design, can be cut in any length by hand, meanwhile keep appropriate tensile strength of the tape.
- Thin, flexible, painless to remove and not damage the skin.
- Good stickness, waterproof, no edge warped.
- Single-use design to maintain hygiene.

DIRECTIONS FOR USE

1. Ensure that the skin is clean, dry, and free of ointment or cream before applying.
2. Choose the proper length of Rystora Silicone Scar Tape through tearing line to make sure it could cover the scar area completely.
3. Slightly press the Rystora Silicone Scar Tape, so that this product can stick to the skin firmly.

SPECIFICATION

REF. No.	Description
SAR251500	25mm x 1.5m Non-Sterile Disposable

Rystora®

Fortune Medical Instrument Corp.

6 F., No. 29, Sec. 2, Jhongjheng E. Rd., Danshuei Dist,
New Taipei City 251, Taiwan

Manufactured by: No. 256, Changchun 2nd, Rd.,
Jhongli Dist, Taoyuan City 320, Taiwan

T. +886 2 2624-2233 www.rystora.com

Rystora® is Registered Trademark of Fortune Medical.

EDM TAPE Eng. V20210625